

AnyWHERE Swing

by Cecilia Cruse, MS, OTR/L

Introduction

Congratulations on your purchase of the AnyWHERE Swing! The mother of a child who has cerebral palsy wanted a safe way to allow her child to swing, and so she designed this great product. This portable device, made of durable nylon, allows the conversion of any standard swing seat to a sling swing with back support. Swinging, as part of a good sensory diet that includes vestibular input, promotes calming and organization of the sensory system. Now, almost any child can benefit from and experience the simple joy of swinging. The AnyWHERE Swing is perfect for children with cerebral palsy or low muscle tone, or any situation where a regular swing is just not enough.

Product Use

It's easy to put up your AnyWHERE Swing:

- Thread the 2 white PVC pipes into the inside baffle of the AnyWHERE swing's back (fig. 1).
- Unhook a standard swing or belt swing from the top of the swing set, keeping the chains attached to the swing seat.
- Slide one end of the standard/belt swing through the bottom opening of the AnyWHERE swing. Pull through until the standard/belt swing is centered inside the seat base of the AnyWHERE swing fabric (fig. 2).
- Thread the chains up through the outside baffle on the high or back part of the swing, and re-attach the chains to the top of the swing set (fig. 3).
- Fasten the child into the AnyWHERE swing using the chest and seat belt.



fig 1. Insert PVC pipes into sides.



fig 2. Slide swing bottom through with chains still attached.

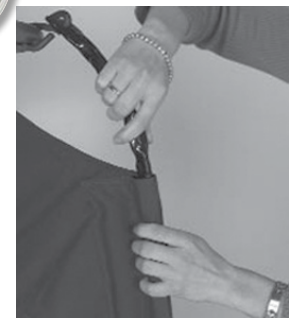


fig 3. Thread chains through outside baffle.

Care & Safety

The AnyWHERE swing is made of nylon fabric that measures 26" x 12" (66 x 30.5 cm) with PVC piping to reinforce the sides, and 1" (2.5 cm) webbing for seat and chest belts. The AnyWHERE swing repels moisture and is durable enough to withstand the elements, but we recommend bringing it inside when not in use. Maximum user weight is 150 lbs (68 kg). Children should be under adult supervision when using the AnyWHERE swing.



©2013 School Specialty • PO Box 1579, Appleton, WI 54912-1579 • USA • All rights reserved.

Customer Service 1-888-388-3224 • www.schoolspecialty.com

Or reach us via e-mail at: orders@schoolspecialty.com

1-YEAR GUARANTEE OF SATISFACTION

See more fun activity guides for other great  products at www.abilitations.com.

