

Introduction

Welcome to the Abilitations Chair Hugger, the device that holds them tight and seats them right! The Chair Hugger was designed with special needs children in mind. Made of soft, stretchy lycra, the Chair Hugger is the perfect assist for providing extra trunk support as well as sensory input. Wonderful for low muscle tone, fidgeting and/or children with sensory processing disorders who require deep pressure for calming and organizing.

Product Use

Designed to be used as an assist with sitting in most standard chair types. Be sure the chair provides a very stable base. Place the child in the chair and loop the Chair Hugger around the child's trunk keeping the lycra fabric as fully extended as possible. Secure the Chair Hugger to the back of the chair with a knot.

Care & Safety

The Chair Hugger should not be used as a restraining device. Use with adult supervision at all times. Recommended wearing schedule not to exceed 30 minutes at one time and/ or use with therapist recommendations. The Chair Hugger is a rectangle of lycra fabric that measures 17"W x 53"L (43cm x 134cm). Colors may vary. Machine wash on gentle cycle and hang dry or tumble dry on low.

Recommended Products

If you like our Chair Hugger, check out these other great Abilitations products:

• Abilitations Cuddle Loop: 030940



©2013 School Specialty • PO Box 1579, Appleton, WI 54912-1579 • USA • All rights reserved.

Customer Service 1-888-388-3224 • www.schoolspecialty.com

Or reach us via e-mail at: orders@schoolspecialty.com

1-YEAR GUARANTEE OF SATISFACTION

See more fun activity guides for other great Abilitations.com.